

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat No. CG5022 **Lot No.** 179CG
Size: 12 x 1 ml **Expiry:** 2018-02

INTENDED USE

This product is intended for *in vitro* diagnostic use, and in the quality control of coagulation systems. The Coagulation Controls are for the control of accuracy and precision.

DEVICE DESCRIPTION

The Coagulation Controls are supplied at 3 levels, level 1, 2 and 3. Target values and ranges are supplied for the analytes listed in the values section.

SAFETY PRECAUTIONS AND WARNINGS

The controls are intended for *in vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents and controls.

Human source material, which has been added, has been tested at donor level for the Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and was found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests.

However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and disposed of accordingly.

Health and Safety Data sheets are available on request.

STORAGE AND STABILITY

OPENED: Store refrigerated (+2°C to +8°C). APTT, TT, PT, Fibrinogen and Antithrombin III in reconstituted serum are stable for 24 hours at +2°C to +8°C if kept capped in original container and free from contamination. Protein C, Protein S, Plasminogen, and Factors II, V, VII, VIII, IX, X, XI, XII are stable for 8 hours at +2°C to +8°C. Only the required amount of product should be removed. After use, any residual product should NOT BE RETURNED to the original vial.

UNOPENED: Store refrigerated (+2°C to +8°C). Stable to expiration date printed on individual vials.

PREPARATION FOR USE

The Coagulation Controls are supplied lyophilised.

1. Carefully reconstitute each vial of lyophilised control with exactly 1 ml of distilled water at +15°C to +25°C. Close the bottle and allow to stand for 30 minutes before use. Ensure contents are completely dissolved by swirling gently. Avoid the formation of foam. Do not shake the vial.
2. Refer to the Control section of the individual analyser application.
3. Refrigerate any unused material. Prior to reuse, mix the contents of the vial thoroughly.

MATERIALS PROVIDED

Coagulation Control - Level 2 12 x 1 ml

MATERIALS REQUIRED BUT NOT PROVIDED

Volumetric pipette

ASSIGNED VALUES

Each batch of Coagulation Control is submitted to a number of external laboratories. Values are assigned from a consensus of results obtained by these laboratories and internal testing conducted at Randox Laboratories Ltd. The expected range of the mean is provided to aid laboratory, until it has established its own mean and SD for its test methods.

04 Dec '15 ne

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG

Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Activated Partial Thromboplastin Time (APTT)	seconds	67.5	54.0	81.0	Tcoag TriniCLOT aPTT S
	seconds	77.2	56.2	98.2	Tcoag TriniCLOT aPTT HS
	seconds	84.2	67.4	101	Siemens/Dade Actin
	seconds	63.6	49.9	77.3	Siemens/Dade Actin FS
	seconds	58.4	45.1	71.7	Siemens/Dade Actin FSL
	seconds	117	93.6	140	Siemens/Dade Actin SL
	seconds	116	92.8	139	Siemens/Dade Pathromtin SL
	seconds	75.3	58.9	91.7	Helena aPTT
	seconds	66.3	53.0	79.6	HemosIL APTT-SP Liquid
	seconds	63.5	50.8	76.2	HemosIL Synthasil
	seconds	61.7	49.4	74.0	HemosIL Synthafax
	seconds	58.6	46.9	70.3	Pacific Hemostasis APTT-XL
	seconds	77.1	57.0	97.2	Stago aPTT
	seconds	68.4	54.7	82.1	Stago Cephascreen
	seconds	74.5	58.6	90.4	Stago CK-Prest
	seconds	85.1	68.1	102	Stago PTT Automate
	seconds	71.8	55.4	88.2	Teco TEClot APTT
	seconds	65.0	48.1	81.9	Bio-Ksel System APTTs
	seconds	79.0	58.7	99.3	Diagon Dia-PTT
	seconds	69.5	55.6	83.4	Weiner Lab APTT
	seconds	74.9	59.9	89.9	Biolabo Bio-CK APTT
	seconds	48.1	38.5	57.7	Vital Diagnostics ElectaLab APTT
	seconds	83.2	62.5	104	Tcoag TriniCLOT Automated aPTT
	seconds	88.9	71.1	107	Biodevice aPTT
	seconds	57.4	45.9	68.9	Biomedica QuikCoag aPTT
	seconds	86.9	68.5	105	Siemens Dade Actin Cephaloplastin
	seconds	57.3	45.8	68.8	Technoclone Siron aPTT
	seconds	69.0	55.0	83.0	Sclavo APTT-S
	seconds	71.8	57.4	86.2	Tokra Medikal APTT
	seconds	56.8	45.4	68.2	Coagpia APTT-N
Activated Partial Thromboplastin Time (APTT) Ratio	ratio	2.18	1.70	2.66	Siemens/Dade Actin FSL
	ratio	3.93	3.14	4.72	Siemens/Dade Actin SL
	ratio	2.25	1.80	2.70	Stago Cephascreen
	ratio	2.33	1.86	2.80	Stago PTT Automate
	ratio	2.30	1.84	2.76	Stago aPTT
	ratio	3.59	2.87	4.31	Siemens/Dade Pathromtin SL
	ratio	2.18	1.74	2.62	HemosIL APTT-SP Liquid
	ratio	2.07	1.66	2.48	HemosIL Synthasil
	ratio	1.84	1.47	2.21	Pacific Hemostasis APTT-XL
	ratio	2.35	1.86	2.84	Stago CK-Prest
	ratio	2.19	1.75	2.63	Bio-Ksel System APTTs
	ratio	2.38	1.90	2.86	Siemens/Dade Actin FS

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG

Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Activated Partial Thromboplastin Time (APTT) Ratio	ratio	2.38	1.90	2.86	Diagon Dia-PTT
	ratio	2.42	1.94	2.90	Biolabo Bio-CK APTT
	ratio	2.73	2.18	3.28	Siemens Dade Actin Cephaloplastin
	ratio	2.37	1.79	2.95	Sclavo APTT-S
	ratio	3.03	2.42	3.64	HemoDiagnostica aPTT
	ratio	1.75	1.35	2.15	Coagpia APTT-N
Antithrombin III activity	%	63.7	51.0	76.4	Siemens/Dade Berichrom ATIII
	%	60.5	48.4	72.6	Helena Chrom-Z
	%	64.7	51.8	77.6	HemosIL Liquid Antithrombin
	%	65.5	52.4	78.6	Roche Chronogenic Thrombin
	%	67.1	53.7	80.5	Hemosil Lyophilised Antithrombin III
	%	66.5	53.2	79.8	STACHrom ATIII
	%	63.5	50.8	76.2	Siemens/Dade Antithrombin III
	%	70.5	56.4	84.6	Biodevice ATIII
	%	66.7	53.3	80.1	Siemens Innovance Antithrombin III
Factor II activity	%	64.1	51.3	76.9	Sclavo ATIII
	%	21.9	17.5	26.3	Dade Innovin
	%	22.9	18.3	27.5	Siemens/Dade Thromborel S
	%	19.8	15.8	23.8	Stago Neoplastine CI Plus
	%	19.5	15.6	23.4	HemosIL PT-Fibrinogen HS Plus
	%	17.7	13.8	21.6	HemosIL PT-Fibrinogen Recombiplastin 2G
Factor IX activity	%	18.0	14.4	21.6	Stago Neoplastin R
	%	21.7	17.4	26.0	Stago PTT Automate
	%	21.4	17.1	25.7	Siemens/Dade Actin FS
	%	25.2	20.2	30.2	Siemens/Dade Pathromtin SL
	%	19.6	15.7	23.5	HemosIL Synthasil
	%	24.5	19.6	29.4	Stago CK-Prest
	%	22.4	17.5	27.3	Siemens/Dade Actin FSL
	%	20.5	16.4	24.6	HemosIL APTT-SP Liquid
Factor V activity	%	24.8	19.8	29.8	Stago aPTT
	%	28.8	23.0	34.6	Siemens Dade Actin Cephaloplastin
	%	39.1	31.3	46.9	Stago Neoplastine CI Plus
	%	40.5	30.9	50.1	Siemens/Dade Thromborel S
	%	36.6	29.3	43.9	Dade Innovin
	%	43.0	34.4	51.6	Stago Neoplastine CI
	%	40.1	29.6	50.6	HemosIL PT-Fibrinogen HS Plus
	%	40.4	31.4	49.4	HemosIL PT-Fibrinogen Recombiplastin 2G
Factor VII activity	%	37.8	30.2	45.4	Stago Neoplastin R
	%	43.2	34.6	51.8	HemosIL Factor V deficient plasma
	%	21.6	17.3	25.9	Stago Neoplastine CI Plus
	%	23.3	18.6	28.0	Siemens/Dade Thromborel S
	%	25.1	20.1	30.1	Dade Innovin
	%	23.7	19.0	28.4	HemosIL PT-Fibrinogen HS Plus
	%	23.1	18.5	27.7	HemosIL PT-Fibrinogen Recombiplastin 2G
	%	22.0	17.6	26.4	Stago Neoplastin R
	%	24.7	19.5	29.9	HemosIL Factor VII deficient plasma

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG

Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Factor VIII activity	%	39.3	31.4	47.2	Stago CK-Prest
	%	42.2	33.8	50.6	Stago PTT Automate
	%	41.3	33.0	49.6	Siemens/Dade Actin FS
	%	40.0	31.6	48.4	Siemens/Dade Pathromtin SL
	%	39.6	31.4	47.8	Siemens/Dade Actin FSL
	%	37.0	27.6	46.4	HemosIL Synthasil
	%	39.9	30.7	49.1	HemosIL APTT-SP Liquid
	%	35.8	28.6	43.0	Stago aPTT
	%	40.7	32.6	48.8	Siemens Dade Actin Cephaloplastin
	%	41.9	33.5	50.3	HemosIL Factor VIII deficient plasma
Factor X activity	%	24.2	17.9	30.5	Siemens/Dade Thromborel S
	%	22.8	18.2	27.4	Dade Innovin
	%	21.1	16.6	25.6	HemosIL PT-Fibrinogen HS Plus
	%	20.2	16.2	24.2	Stago Neoplastine CI Plus
	%	18.5	13.1	23.9	HemosIL PT-Fibrinogen Recombiplastin 2G
	%	19.6	15.2	24.0	Stago Neoplastin R
Factor XI activity	%	23.0	18.4	27.6	Stago PTT Automate
	%	25.2	19.1	31.3	Siemens/Dade Actin FS
	%	26.2	21.0	31.4	Siemens/Dade Pathromtin SL
	%	20.6	14.4	26.8	HemosIL Synthasil
	%	22.5	18.0	27.0	Siemens/Dade Actin FSL
	%	22.9	18.3	27.5	HemosIL APTT-SP Liquid
Factor XII activity	%	45.9	36.7	55.1	Siemens/Dade Actin FS
	%	48.1	38.5	57.7	Siemens/Dade Pathromtin SL
	%	47.6	38.1	57.1	HemosIL Synthasil
	%	49.0	39.2	58.8	Siemens/Dade Actin FSL
	%	49.6	39.7	59.5	HemosIL APTT-SP Liquid
	%	54.9	43.9	65.9	Siemens Dade Actin Cephaloplastin
Fibrinogen	g/L	2.28	1.82	2.74	HemosIL PT-Fibrinogen HS Plus
	mg/dL	228	182	274	
	g/L	1.86	1.49	2.23	Stago Fibrinogen
	mg/dL	186	149	223	
	g/L	2.09	1.67	2.51	Tcoag TriniCLOT Thrombin Time
	mg/dL	209	167	251	
	g/L	2.11	1.69	2.53	Dade Multifibrin
	mg/dL	211	169	253	
	g/L	1.93	1.54	2.32	HemosIL Fibrinogen C
	mg/dL	193	154	232	
	g/L	1.72	1.38	2.06	Dade Thrombin
	mg/dL	172	138	206	
	g/L	1.92	1.54	2.30	Technoclone Fibrinogen
	mg/dL	192	154	230	
	g/L	1.91	1.53	2.29	Tcoag TriniCLOT Fibrinogen
	mg/dL	191	153	229	
	g/L	1.87	1.50	2.24	Biolabo Bio-Fibri
	mg/dL	187	150	224	

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG

Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Fibrinogen	g/L	1.73	1.38	2.08	Teco Teclot Fibrinogen
	mg/dL	173	138	208	
	g/L	1.80	1.44	2.16	HemosIL QFA (bovine thrombin)
	mg/dL	180	144	216	
	g/L	1.76	1.41	2.11	Pacific Hemostasis Fibrinogen
	mg/dL	176	141	211	
	g/L	1.97	1.58	2.36	Human Fibrinogen
	mg/dL	197	158	236	
	g/L	1.85	1.48	2.22	Stago STA Fibri-Prest Automate
	mg/dL	185	148	222	
	g/L	2.39	1.91	2.87	Bio-Ksel System PT
	mg/dL	239	191	287	
	g/L	1.69	1.34	2.04	Bio-Ksel Fibrinogen
	mg/dL	169	134	204	
	g/L	1.80	1.44	2.16	Diagon Dia-FIB
	mg/dL	180	144	216	
	g/L	1.87	1.48	2.26	Dade Innovin Derived Fibrinogen
	mg/dL	187	148	226	
	g/L	1.89	1.51	2.27	Siemens/Dade Thromboplastin Derived Fibrinogen
	mg/dL	189	151	227	
	g/L	2.22	1.78	2.66	Wiener Lab Fibrinogen
	mg/dL	222	178	266	
	g/L	1.81	1.45	2.17	Liofilchem Fibrinogen
	mg/dL	181	145	217	
	g/L	1.58	1.26	1.90	SGM Italia Fibrinogen
	mg/dL	158	126	190	
	g/L	1.94	1.52	2.36	Futura System Fibrinogen
	mg/dL	194	152	236	
	g/L	2.05	1.51	2.59	Biodevice PT
	mg/dL	205	151	259	
	g/L	2.16	1.67	2.65	HemosIL PT-Fibrinogen Recombiplastin 2G
	mg/dL	216	167	265	
	g/L	1.90	1.52	2.28	Sclavo Fibrinogen
	mg/dL	190	152	228	
	g/L	1.95	1.56	2.34	HemoDiagnostica Fibrinogen
	mg/dL	195	156	234	
	g/L	1.85	1.48	2.22	Beijing Steelex Fibrinogen
	mg/dL	185	148	222	
	g/L	1.56	1.25	1.87	Cypress Diagnostics Fibrinogen
	mg/dL	156	125	187	
	g/L	1.92	1.54	2.30	Helena Clauss Fibrinogen 100
	mg/dL	192	154	230	
	g/L	1.95	1.56	2.34	Coagpia Fibrinogen
	mg/dL	195	156	234	
Plasminogen activity	%	69.3	55.4	83.2	Stago Stachrom Plasminogen
	%	65.0	52.0	78.0	HemosIL Plasminogen

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Plasminogen activity	%	67.6	54.0	81.2	Siemens/Dade Berichrom Plasminogen
Protein C activity	%	22.6	18.1	27.1	Siemens/Dade Berichrom Protein C
	%	20.5	16.3	24.7	Stago StaClot
	%	18.0	14.4	21.6	IL HemosIL Protein C
	%	24.5	19.6	29.4	Siemens
	%	22.9	18.3	27.5	STACHrom Protein C
	%	23.1	18.5	27.7	Chromogenix Coatest APC
Protein S activity	%	16.5	11.6	21.5	STA Liatest
	%	16.6	11.6	21.6	Helena Elisa
	%	13.1	9.17	17.0	IL HemosIL ProS
	%	15.7	11.0	20.4	IL HemosIL Free Protein S
	%	15.3	10.7	19.9	Stago Asserachrom Free Protein S
	%	16.4	11.5	21.3	Siemens Innovance Free PS Ag
Prothrombin Time (PT)	Seconds	19.3	15.4	23.2	Tcoag TriniCLOT PT Excel
	Seconds	25.6	20.5	30.7	Tcoag TriniCLOT PT HTF
	Seconds	20.8	16.6	25.0	Dade Innovin
	Seconds	27.2	21.8	32.6	Siemens/Dade Thromborel S
	Seconds	32.4	25.9	38.9	HemosIL PT-Fibrinogen HS Plus
	Seconds	37.7	26.6	48.8	Pacific Thromboplastin DS
	Seconds	29.1	23.3	34.9	STA Hepato Quick
	Seconds	28.3	20.7	35.9	Stago Neoplastine CI
	Seconds	30.0	24.0	36.0	Stago Neoplastine CI Plus
	Seconds	24.5	19.6	29.4	R2 Phosphoplastin RL
	Seconds	28.6	22.9	34.3	Teco TEClot PT-S
	Seconds	30.9	24.7	37.1	Biolabo Bio-TP
	Seconds	34.9	27.9	41.9	Stago Neoplastin R
	Seconds	23.0	18.4	27.6	Bio-Ksel System PT
	Seconds	28.9	23.1	34.7	Human Thromboplastin
	Seconds	30.3	24.2	36.4	Grifols DG-PT
	Seconds	26.4	20.2	32.6	Diagon, Dia-PT
	Seconds	27.4	21.9	32.9	Weiner Lab Soluplastin
	Seconds	31.5	24.7	38.3	Spinreact PT
	Seconds	34.5	27.6	41.4	TCoag TriniCLOT PT Excel S
	Seconds	26.2	21.0	31.4	Biodevice PT
	Seconds	27.4	21.6	33.2	Biomedica QuikCoag PT-HS
	Seconds	25.6	20.5	30.7	HemosIL PT-Fibrinogen Recombiplastin 2G
	Seconds	28.2	22.6	33.8	Helena Thromboplastin LI
	Seconds	25.5	19.5	31.5	Beijing Steellex PT
	Seconds	29.6	23.7	35.5	Labitec PT
	Seconds	26.9	21.5	32.3	SGM Italia Thromboplastin
	Seconds	25.2	20.2	30.2	Technoclone Technoclot PT Plus
	Seconds	29.2	23.4	35.0	Coagpia PT-N
	Seconds	23.8	19.0	28.6	Bio-Ksel PT Plus
	Seconds	26.0	20.8	31.2	Beijing Succeeder PT
Prothrombin Time (PT) activity	%	32.6	26.1	39.1	Tcoag TriniCLOT PT Excel
	%	24.1	19.3	28.9	Tcoag TriniCLOT PT HTF

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Prothrombin Time (PT) activity	%	33.3	26.6	40.0	Dade Innovin
	%	29.5	23.6	35.4	Siemens/Dade Thromborel S
	%	30.8	24.6	37.0	Stago Neoplastine CI
	%	27.3	21.8	32.8	Stago Neoplastine CI Plus
	%	27.7	22.2	33.2	HemosIL PT-Fibrinogen HS Plus
	%	26.2	21.0	31.4	Biolabo Bio-TP
	%	20.1	15.2	25.0	Stago SPA
	%	27.8	22.2	33.4	Stago Neoplastin R
	%	45.3	36.2	54.4	Bio-Ksel System PT
	%	27.6	19.9	35.3	Human Thromboplastin
	%	36.1	28.9	43.3	Diagon, Dia-PT
	%	27.7	22.2	33.2	TCoag TriniCLOT PT Excel S
	%	34.6	27.7	41.5	Diagen Thromboplastin S
	%	33.6	26.9	40.3	Biomedica QuikCoag PT-HS
	%	31.2	25.0	37.4	HemosIL PT-Fibrinogen Recombiplastin 2G
	%	29.6	23.7	35.5	Labitec PT
	%	33.3	26.6	40.0	Technoclone Technoclot PT Plus
	%	23.8	19.0	28.6	Coagpia PT-N
	%	45.0	36.0	54.0	Bio-Ksel PT Plus
Prothrombin Time (PT) INR	INR	2.37	1.90	2.84	Tcoag TriniCLOT PT HTF
	INR	1.96	1.57	2.35	Dade Innovin
	INR	2.48	1.98	2.98	Siemens/Dade Thromborel IS
	INR	2.37	1.90	2.84	Siemens/Dade Thromborel S
	INR	2.66	2.13	3.19	HemosIL PT-Fibrinogen HS Plus
	INR	2.96	2.29	3.63	Pacific Thromboplastin DS
	INR	2.89	2.09	3.69	STA Hepato Quick
	INR	2.70	2.11	3.29	Stago Neoplastine CI
	INR	2.86	2.29	3.43	Stago Neoplastine CI Plus
	INR	2.09	1.67	2.51	R2 Phosphoplastin RL
	INR	2.46	1.97	2.95	Human Thromboplastin
	INR	2.27	1.82	2.72	Teco TEClot PT-S
	INR	2.98	2.38	3.58	Biolabo Bio-TP
	INR	2.56	2.05	3.07	Stago SPA
	INR	2.42	1.94	2.90	Stago Neoplastin R
	INR	2.22	1.78	2.66	Bio-Ksel System PT
	INR	2.64	2.11	3.17	Grifols DG-PT
	INR	2.33	1.74	2.92	Diagon, Dia-PT
	INR	2.48	1.78	3.18	Weiner Lab Soluplastin
	INR	2.61	2.03	3.19	Futura System Thromboplastin S
	INR	2.32	1.86	2.78	SGM Italia Thromboplastin
	INR	2.80	2.24	3.36	TCoag TriniCLOT PT Excel S
	INR	2.26	1.81	2.71	Media IVD PT
	INR	2.06	1.65	2.47	Biodevice PT
	INR	2.33	1.78	2.88	Biomedica QuikCoag PT-HS
	INR	2.29	1.83	2.75	HemosIL PT-Fibrinogen Recombiplastin 2G
	INR	2.33	1.86	2.80	Helena Thromboplastin LI

COAGULATION CONTROL - LEVEL 2 (COAG CONTROL 2)

Cat. No. CG5022 Lot No. 179CG Size: 12 x 1 ml Expiry: 2018-02

Range					
Analyte	unit	Target	low	high	methods
Prothrombin Time (PT) INR	INR	1.95	1.56	2.34	Technoclone Technoclot PT Plus
	INR	2.38	1.90	2.86	Coagpia PT-N
	INR	2.23	1.78	2.68	Bio-Ksel PT Plus
	INR	2.12	1.70	2.54	Beijing Succeeder PT
Prothrombin Time (PT) ratio	ratio	2.02	1.62	2.42	Dade Innovin
	ratio	2.35	1.88	2.82	Siemens/Dade Thromborel S
	ratio	2.19	1.75	2.63	Stago Neoplastine CI
	ratio	2.26	1.81	2.71	Stago Neoplastine CI Plus
	ratio	2.32	1.86	2.78	Stago Neoplastin R
	ratio	2.12	1.59	2.65	Diagon, Dia-PT
	ratio	2.34	1.87	2.81	HemosIL PT-Fibrinogen HS Plus
	ratio	2.24	1.79	2.69	HemosIL PT-Fibrinogen Recombiplastin 2G
Thrombin Time (TT)	Seconds	28.8	22.0	35.6	Dade Thrombin
	Seconds	19.2	15.4	23.0	Siemens/Dade Thromboclotin
	Seconds	23.3	18.6	28.0	Stago Thrombin
	Seconds	29.7	23.8	35.6	Sysmex Thrombin Time
	Seconds	27.1	21.7	32.5	Diagon, Dia-TT
	Seconds	22.2	17.8	26.6	Teco TEClot APTT
	Seconds	18.2	14.6	21.8	Technoclone Thrombin Reagent
	Seconds	32.2	24.6	39.8	HemosIL Thrombin Time 5ml