

PRODUCT INFORMATION

IAS3118

2070EC

The reconstituted stability for Immunoassay Speciality II Level 2 lot 2070EC for Renin is 1 day at +2°C to +8°C, or 28 days when frozen once at -20°C.

Please note that values for Plasma Renin Activity are not available for Immunoassay Speciality II Level 2 lot 2070EC.

INC246 / INC297

IMMUNOASSAY SPECIALITY II - LEVEL 2 (IA SPECIALITY II LEV 2)

CAT. NO. IAS 3118

LOT NO. 2070EC

SIZE: 5 x 1 ml

EXPIRY: 2023-02-28

GTIN: 05055273207330

INTENDED USE

This product is intended for *in vitro* diagnostic use, in the quality control of the accuracy of Immunoassays on clinical chemistry systems. This material can be used to monitor the control of accuracy or the control of reproducibility of immunoassays.

SAFETY PRECAUTIONS AND WARNINGS

For *in vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

Human source material, from which this product has been derived, has been tested at donor level for the Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests.

However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and disposed of accordingly.

Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

OPENED: Store refrigerated (+2°C to +8°C). In reconstituted serum Renin is stable for 1 day at +2°C to +8°C, or 28 days when frozen once at -20°C, Procalcitonin is stable for 1 day, Gastrin and Calcitonin are stable for 8 hours at +2°C to +8°C if kept capped in original container and free from contamination. The control is stable if frozen once for 28 days at -20°C, except for the Siemens Advia Centaur CALCT assay (Calcitonin) which is stable for 14 days at -20°C. Only the required amount of product should be removed. After use, any residual product should NOT BE RETURNED to the original vial.

UNOPENED: Store refrigerated (+2°C to +8°C). Stable to expiration date printed on individual vials.

Bacterial contamination of the reconstituted serum will cause reductions in the stability of many components. If bacterial contamination is suspected, the vial should be discarded and a fresh vial reconstituted.

PREPARATION

Immunoassay Speciality II is supplied lyophilised.

1. Carefully reconstitute each vial of lyophilised serum with exactly 1 ml of distilled water at +15°C to +25°C. Close the bottle and allow to stand for 30 minutes before use. Ensure contents are completely dissolved by swirling gently. Avoid formation of foam. Do not shake.
2. Refer to the Control section of the individual analyser application.
3. Refrigerate any unused material. Prior to reuse, mix contents thoroughly.

MATERIALS PROVIDED

Immunoassay Speciality II - Level 2 5 x 1 ml


MATERIAL REQUIRED BUT NOT PROVIDED

Volumetric pipette

VALUE ASSIGNMENT

Each batch of Immunoassay Speciality II is submitted to a number of reference laboratories and values are assigned from a consensus of results obtained by these laboratories, using a unique statistical analysis. With each batch, a control range is provided for individual parameters and each parameter method. The control range is equivalent to the assigned mean ± 2 S.D. This results in extremely accurate values, which may be confidently used by laboratories to ensure the accuracy of their methods.

If a method is unavailable, contact Randox Laboratories - Technical Services, Northern Ireland, tel: +44 (0) 28 9445 1070 or email Technical.Services@randox.com.

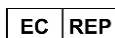

Randox Teoranta, Meenmore,
Dungloe, Donegal,
F94 TV06, Ireland

Rev. 16 Sep '21 me



IMMUNOASSAY SPECIALITY II - LEVEL 2 (IA SPECIALITY II LEV 2)

Cat. No. IAS3118 Lot. No. 2070EC Size 5 x 1ml Expiry 2023-02-28

Analyte	unit	Target	Range		methods
			low	high	
Calcitonin	pmol/l	34.8	26.1	43.5	Siemens Immulite 2000/2500
	pg/ml	119	89.0	149	
	pmol/l	29.3	22.0	36.6	Diasorin Liaison
	pg/ml	100	75.0	125	
	pmol/l	42.3	31.7	52.9	Roche Elecsys/Cobas/Modular
	pg/ml	144	108	180	
	pmol/l	29.1	21.8	36.4	Diasorin Liaison XL
	pg/ml	99.2	74.3	124	
Gastrin	pmol/l	37.7	28.3	47.1	Roche Cobas e801
	pg/ml	129	96.5	162	
Gastrin	pmol/l	44.4	33.3	55.5	Siemens Immulite 2000/2500
	pg/ml	92.5	69.4	116	
Procalcitonin	µg/l	3.83	2.87	4.79	BioMerieux Vidas
	µg/l	2.78	2.09	3.48	Roche Elecsys/Cobas/Modular
	µg/l	2.94	2.21	3.68	Abbott Architect / Alinity Brahms PCT
	µg/l	2.59	1.94	3.24	Roche Cobas e801
	µg/l	4.00	3.00	5.00	Beckman Access PCT
	µg/l	2.20	1.65	2.75	Shanghai I-Reader
	µg/l	2.81	2.11	3.51	Ortho Vitros ECi
Renin	mIU/l	47.9	35.9	59.9	Diasorin Liaison direct Renin
	pg/ml	28.2	21.1	35.3	
	mIU/l	49.9	37.4	62.4	Diasorin Liaison XL direct Renin
	pg/ml	29.3	22.0	36.6	