

LIPID CONTROL - LEVEL I (LPD CONTROL I)

Cat. No. LE266 I **Lot No.** 3064CH
Size: 5 x 3 ml **Expiry:** 2026-04-28
GTIN: 05055273204155

INTENDED USE

This product is intended for *in vitro* use in the quality control of Direct HDL, Direct LDL, Lipoprotein (a), Apolipoprotein A-I, Apolipoprotein B, Cholesterol and Triglyceride methods on clinical chemistry systems.

SAFETY PRECAUTIONS AND WARNINGS

Human source material, from which this product has been derived, has been tested at donor level for the Human Immunodeficiency Virus (HIV1 & HIV2) antibody, Hepatitis B surface antigen (HbsAg) and the Hepatitis C virus (HCV) antibody and found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests. However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting disease. For *in vitro* diagnostic use only.

STORAGE AND STABILITY

Unopened Lipid Control is stable until the expiry date printed on the product label when stored between +2°C and +8°C. Once reconstituted, the components of the serum are stable for 7 days at +2°C to +8°C, and 4 weeks at -20°C when frozen once. The following exceptions apply: LP(a) is stable for 16 weeks at -20°C when frozen once. Values may drop by up to 10% for Direct LDL Cholesterol when stored for 4 weeks at -20°C.

PREPARATION FOR USE

Open the vial carefully, avoiding any loss of the material and reconstitute with 3 ml of distilled water. Replace the rubber stopper, close the vial and leave to stand for 30 minutes before use. Ensure that all traces of dry material are dissolved by swirling gently.

MATERIALS PROVIDED

Lipid Control - Level I 5 x 3 ml

MATERIALS REQUIRED BUT NOT PROVIDED

Distilled water
Volumetric pipette

VALUE ASSIGNMENT

Due to the variation caused by test equipment, test reagents and laboratory technique, the quoted ranges are provided for guidance. It is recommended that these ranges are used until each laboratory has established its own ranges, based on individual laboratory requirements.

Each batch of Lipid Control is submitted to a number of external laboratories. Values are assigned from a consensus of results obtained by these laboratories and internal testing conducted at Randox Laboratories Ltd.

If a method is unavailable, contact Randox Laboratories - Technical Services, Northern Ireland, tel: +44 (0) 28 9445 1070 or email Technical.Services@randox.com.

EC	REP	Randox Teoranta, Meenmore, Dungloe, Donegal, F94 TV06, Ireland
----	-----	--

08 Sep '23 me

LIPID CONTROL - LEVEL 1 (LPD CONTROL 1)

Lot. No. 3064CH Cat. No. LE2661

Size 5 x 3 ml Expiry 2026-04-28

Range

Analyte	unit	Target	low	high	methods
Apolipoprotein A-1	g/l	1.05	0.86	1.24	Immunoturbidimetric
	mg/dl	105	86.1	124	
	g/l	1.07	0.88	1.26	Nephelometric
	mg/dl	107	87.7	126	
Apolipoprotein B	g/l	0.88	0.72	1.04	Immunoturbidimetric
	mg/dl	88.1	72.2	104	
	g/l	0.85	0.69	1.00	Nephelometric
	mg/dl	84.6	69.4	100	
Cholesterol	mmol/l	4.26	3.71	4.81	Cholesterol Oxidase - Abell Kendall
	mg/dl	164	143	185	
	mmol/l	4.21	3.66	4.76	Siemens Dimension
	mg/dl	163	141	185	
	mmol/l	4.26	3.71	4.81	Cholesterol Oxidase - IDMS
	mg/dl	164	143	185	
HDL - Cholesterol	mmol/l	0.94	0.80	1.09	Direct Clearance Method
	mg/dl	36.4	31.0	41.8	
	mmol/l	0.48	0.41	0.55	Phosphotungstic acid pptn.
	mg/dl	18.5	15.7	21.3	
	mmol/l	0.91	0.77	1.04	Direct HDL Immunoseparation
	mg/dl	34.9	29.7	40.1	
	mmol/l	0.99	0.84	1.14	Direct HDL PEGME
	mg/dl	38.3	32.6	44.0	
	mmol/l	1.04	0.88	1.20	Direct HDL PPD
	mg/dl	40.1	34.1	46.1	
mmol/l	0.87	0.74	1.00	Direct HDL Roche 4th Generation	
mg/dl	33.7	28.6	38.8		
LDL - Cholesterol	mmol/l	1.03	0.88	1.18	HDL - Ultra
	mg/dl	39.8	34.0	45.6	
	mmol/l	2.52	2.14	2.90	Direct Clearance Method
	mg/dl	97.3	82.6	112	
	mmol/l	2.59	2.20	2.98	Calculated
	mg/dl	100	84.9	115	
mmol/l	2.34	1.99	2.69	Selective detergent methods	
mg/dl	90.3	76.8	104		
Lipoprotein (a)	mg/dl	18.5	14.8	22.2	Immunoturbidimetric
	nmol/l	33.9	27.1	40.7	
Triglycerides	mmol/l	1.53	1.29	1.77	Lipase/GPO-PAP no correction
	mg/dl	135	114	156	
	mmol/l	1.64	1.38	1.90	Lipase/Glycerol Dehydrogenase
	mg/dl	145	122	168	

LIPID CONTROL - LEVEL 1 (LPD CONTROL 1)

Lot. No. 3064CH Cat. No. LE2661

Size 5 x 3 ml Expiry 2026-04-28

Range

Analyte	unit	Target	low	high	methods
Triglycerides	mmol/l	1.57	1.32	1.82	Lipase/GK UV no correction
	mg/dl	139	117	161	