

PRODUCT INFORMATION

HE1532

1359UE

Please be aware that values are currently not available on control catalogue number HE1532 lot 1359UE. Targets and ranges will be available in due course.

For any questions or queries please contact technical.services@randox.com

CCS INC667

HUMAN ASSAYED MULTI-SERA - LEVEL 3 (HUM ASY CONTROL 3)

CAT. NO.	HE1532	GTIN:	05055273203608	SIZE	20 x 5ml
CAT. NO.	HS2611	GTIN:	05055273203813	SIZE	5 x 5ml
LOT NO.	1359UE	EXPIRY:	2027-12-28		

INTENDED USE

This product is intended for *in vitro* diagnostic use, in the quality control of diagnostic assays. The Human Assayed Multi-sera is for the control of accuracy.

DEVICE DESCRIPTION

The Human Assayed Multi-sera is supplied at 2 levels, level 2 and 3. Target values and ranges are supplied for the analytes listed in the values section at both levels.

SAFETY PRECAUTIONS AND WARNINGS

For *in vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

Human source material, from which this product has been derived, has been tested at donor level for the Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests.

However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and disposed of accordingly.

Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

OPENED: Store refrigerated (+2°C to +8°C). Reconstituted serum is stable for 8 hours at +15°C to +25°C or 7 days at +2°C to +8°C, and 28 days when frozen once at -18°C to -24°C. (See Limitations)

UNOPENED: Store refrigerated (+2°C to +8°C). Stable to expiration date printed on individual vials.

LIMITATIONS

For Total & Prostatic Acid Phosphatase, the material should be stabilised by adding 1 drop (25µl - 30µl) of 0.7M Acetic acid solution to 1ml of the serum exactly 30 minutes after reconstitution. After stabilisation Total and Prostatic Acid Phosphatase is stable for 2 hours at +15°C to +25°C, 2 days at +2°C to +8°C, and 28 days when frozen once at -18°C to -24°C.

Alkaline Phosphatase levels in the reconstituted serum will rise over the stability period. It is recommended that the reconstituted serum is allowed to stand for 1 hour at +15°C to +25°C before measurement.

Bilirubin in the serum is light sensitive and it is recommended that the serum is stored in the dark. Stored in the dark, it is stable for 4 days at +2°C to +8°C. Do not store at +15°C to +25°C. Do not freeze.

GLDH is stable for 2 days at +2°C to +8°C.

NEFA is stable for 1 day at +2°C to +8°C.

Total PSA is stable for 4 days at +2°C to +8°C, or 28 days in aliquots frozen at -18°C to -24°C.

Bacterial contamination of the reconstituted serum will cause reductions in the stability of many components.

Different lot numbers of this control should not be interchanged, as the values assigned to the controls vary from lot to lot.

The control should not be used as a calibration material.

Due to the zinc content in some batches of rubber stoppers, the QC and calibrator material should be aliquoted into polypropylene tubes and stored at +2°C to +8°C to ensure stable zinc levels throughout the stability period.

PREPARATION FOR USE

The Human Assayed Multi-sera is supplied lyophilised.

1. Carefully reconstitute each vial of lyophilised serum with exactly 5ml of distilled water at +15°C to +25°C. Close the bottle and allow to stand for 30 minutes before use. Ensure contents are completely dissolved by swirling gently. Avoid formation of foam. Do not shake.
2. Refer to the Control section of the individual analyser application.
3. Refrigerate any unused material. Prior to reuse, mix contents thoroughly.

MATERIALS PROVIDED

Human Assayed Multi-sera - Level 3 20 x 5ml / 5 x 5ml

MATERIALS REQUIRED BUT NOT PROVIDED

Volumetric pipette

ASSIGNED VALUES

Please be aware that values are currently not available for this control. Targets and ranges will be available in due course. For any questions or queries please contact technical.services@randox.com

EC	REP
----	-----

Randox Teoranta, Meenmore,
Dungloe, Donegal,
F94 TV06, Ireland

Rev. 05 Mar '24 me