

## **LIPID CONTROL - LEVEL I (LPD CONTROL I)**

**Cat. No.** LE2668**Lot No.** 2314CH**Size:** 5 x 1 ml**Expiry:** 2017-10-28**GTIN:** 05055273204186

### **INTENDED USE**

This product is intended for *in vitro* use, in the quality control of Direct HDL, Direct LDL, Lipoprotein (a), Apolipoprotein A-I, Apolipoprotein B, Cholesterol and Triglyceride methods on clinical chemistry systems.

### **SAFETY PRECAUTIONS AND WARNINGS**

Human source material, from which this product has been derived, has been tested at donor level for the Human Immunodeficiency Virus (HIV1 & HIV2) antibody, Hepatitis B surface antigen (HbsAg) and the Hepatitis C virus (HCV) antibody and found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests. However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting disease. For *in vitro* diagnostic use only.

### **STORAGE AND STABILITY**

Unopened Lipid Control is stable until the expiry date printed on the product label, when stored between +2°C and +8°C. Once reconstituted, the components of the serum are stable for 7 days at +2°C to +8°C, and 4 weeks at -20°C when frozen once. The following exceptions apply: LP(a) is stable for 16 weeks at -20°C when frozen once. Values may drop by up to 10% for Direct LDL Cholesterol when stored for 4 weeks at -20°C.

### **PREPARATION FOR USE**

Open the vial carefully, avoiding any loss of the material and reconstitute with 1 ml of distilled water. Replace the rubber stopper, close the vial and leave to stand for 30 minutes before use. Ensure that all traces of dry material are dissolved by swirling gently.

### **MATERIALS PROVIDED**

Lipid Control - Level I 5 x 1 ml

### **MATERIALS REQUIRED BUT NOT PROVIDED**

Distilled water

Volumetric pipette

### **VALUE ASSIGNMENT**

Each batch of Lipid Control is submitted to a number of external laboratories. Values are assigned from a consensus of results obtained by these laboratories and internal testing conducted at Randox Laboratories Ltd.

If a method is unavailable, contact Randox Laboratories - Technical Services, Northern Ireland, tel: +44 (0) 28 9445 1070 or email [Technical.Services@randox.com](mailto:Technical.Services@randox.com).

Rev. 30 Aug '16 ne

## LIPID CONTROL - LEVEL 1 (LPD CONTROL 1)

Cat. No. LE2668 Lot No. 2314CH

Size: 5 x 1 ml Expiry: 2017-10-28

Range

Analyte	unit	Target	low	high	methods
Apolipoprotein A-1	g/l	1.16	0.95	1.37	Immunoturbidimetric
	mg/dl	116	95.1	137	
	g/l	1.02	0.84	1.20	Nephelometric
	mg/dl	102	83.6	120	
Apolipoprotein B	g/l	0.89	0.73	1.05	Immunoturbidimetric
	mg/dl	88.8	72.8	105	
	g/l	0.92	0.75	1.08	Nephelometric
	mg/dl	91.8	75.3	108	
Cholesterol	mmol/l	4.09	3.56	4.62	Cholesterol Oxidase
	mg/dl	158	137	179	
	mmol/l	4.04	3.51	4.57	Siemens Dimension
	mg/dl	156	135	177	
	mmol/l	3.84	3.34	4.34	Ortho Vitros Microslide Systems
	mg/dl	148	129	167	
HDL - Cholesterol	mmol/l	0.77	0.66	0.89	Direct Clearance Method
	mg/dl	29.7	25.3	34.1	
	mmol/l	0.38	0.33	0.44	Phosphotungstic acid pptn.
	mg/dl	14.7	12.5	16.9	
	mmol/l	0.86	0.73	0.99	Direct HDL Immunoseparation
	mg/dl	33.2	28.2	38.2	
	mmol/l	1.00	0.85	1.15	Direct HDL PEGME
	mg/dl	38.6	32.8	44.4	
	mmol/l	0.92	0.78	1.06	Direct HDL PPD
	mg/dl	35.5	30.1	40.9	
	mmol/l	0.93	0.79	1.07	Direct HDL Roche 3rd generation
	mg/dl	35.9	30.5	41.3	
	mmol/l	0.93	0.79	1.07	HDL - Ultra
	mg/dl	35.9	30.5	41.3	
	mmol/l	1.02	0.87	1.17	Vitros dHDL PTA/MgCl <sub>2</sub> direct precipitation
	mg/dl	39.4	33.6	45.2	
LDL - Cholesterol	mmol/l	1.97	1.67	2.27	Direct Clearance Method
	mg/dl	76.0	64.5	87.5	
	mmol/l	2.25	1.91	2.59	Selective detergent methods
	mg/dl	86.9	73.7	100	
Lipoprotein (a)	mmol/l	2.63	2.24	3.02	Calculated
	mg/dl	102	86.5	118	
Lipoprotein (a)	mg/dl	13.0	10.4	15.6	Immunoturbidimetric
	nmol/l	30.1	24.1	36.1	
Triglycerides	mmol/l	1.46	1.23	1.69	Lipase/GPO-PAP no correction
	mg/dl	129	109	149	

## LIPID CONTROL - LEVEL 1 (LPD CONTROL 1)

Cat. No. LE2668 Lot No. 2314CH

Size: 5 x 1 ml Expiry: 2017-10-28

Range

Analyte	unit	Target	low	high	methods
Triglycerides	mmol/l	1.53	1.29	1.77	Ortho Vitros Microslide Systems
	mg/dl	135	114	156	
	mmol/l	1.48	1.24	1.72	Lipase/GK UV no correction
	mg/dl	131	110	152	
	mmol/l	1.42	1.19	1.65	Lipase/GPO-PAP 0.11mmol/l correction
	mg/dl	126	105	147	