

CK-MB CALIBRATOR (CK-MB CAL)

CAT. NO. CK 2393 **LOT NO.** 3865CK
SIZE: 10 x 1 ml **EXPIRY:** 2019-09

SERUM PREPARATION

Open one vial of control serum very carefully, avoiding any loss of material, and reconstitute in exactly 1 ml of redistilled water. Close vial and leave to stand for 15 mins., dissolving contents completely by swirling or rotating gently. CK-MB is stable in the serum for 5 days at +4°C, 8 hours at +25°C and 4 weeks at -20°C when frozen once.

SAFETY

Human source material, from which this product has been derived, has been tested at donor level for the Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE. FDA approved methods have been used to conduct these tests.

However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and disposed of accordingly.

For **IN VITRO** diagnostic use only.

	25°C (U/l)	30°C (U/l)	37°C (U/l)
CK-MB	70	114	197

29 Oct '15 ne

THIS PAGE IS INTENTIONALLY BLANK